Adhesive Resin Cement

ENGLISH

Read all information, precautions and notes before using.

■PRODUCT DESCRIPTION AND GENERAL INFORMATION

ESTECEM (light and self-cure), radiopaque, adhesive resin cement system with excellent handling, esthetic and adhesive properties to teeth and all prosthetic materials. The main components of the ESTECEM KIT are PASTE (A/B) and primers. The PASTE is dispensed in an automatic syringe and is available in various shades (Universal, Clear, White-Spagna). The primer is available in two colors: CLEAR (clear) and ESTELINK (BOND A/BOND B) and TOKUYAMA UNIVERSAL PRIMER (A/B). ESTELINK promotes the adhesion of PASTE to tooth structure. TOKUYAMA UNIVERSAL PRIMER enhances the adhesion of PASTE to prosthetic materials. ESTECEM contains methacrylic acid dimethylolpropane diacrylate (Bis-GMA), Triethylene glycol dimethacrylate (TEGDMA), polyethoxy methacrylate (Bis-MPEPP), Peroxide, Camphorquinone and Silica-Zirconia Filler (Filler Lud 74 w4%). The ESTELINK contains Phosphoric acid monomer, Bisphenol A di(2-hydroxypropoxy) dimethacrylate (Bis-GMA), Triethylene glycol dimethacrylate (TEGDMA), 2-Hydroxyethyl methacrylate (HEMA), Peroxide, Borate catalyst, Acetone, Isopropanol and Purified water. The TOKUYAMA UNIVERSAL PRIMER contains MTU-6 (thiouracil monomer), 4-Methacryloxyethyl methacrylate (Bis-GMA), Bisphenol A di(2-hydroxypropoxy) dimethacrylate (Bis-GMA), Triethylene glycol dimethacrylate (TEGDMA), 1,6-bis (methacryloxyethylcarboxylamino)trimethyl hexane (UDMA), ethanol and acetone.

■INDICATIONS

Cementation of crowns, bridges, inlays, and onlays made of glass/oxide ceramics (porcelain, zirconia and alumina), metal alloys (precious and non-precious) and resin materials including inorganic filler (composite materials). Repair of fractured porcelain fused to metal crowns and all ceramic restorations. Cementation of veneers. Cementation of metal-ceramic bridges. Cementation of metal or resin cores, metal or glass-fiber posts.

■CONTRAINDICATIONS

ESTECEM contains methacrylic monomers, organic solvents and acids. DO NOT use ESTECEM for patients allergic to or hypersensitive to methacrylic monomers, related monomers, organic solvents and acids.

■PRECAUTIONS

- Use ESTELINK and/or TOKUYAMA UNIVERSAL PRIMER as primers for the adhesive surfaces. DO NOT use other primers or adhesives as primer for the adhesive surfaces.
- DO NOT use ESTECEM for any purpose other than those listed in these instructions. Use ESTECEM for restorative and aesthetic purposes.
- ESTECEM is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.
- DO NOT use the ESTECEM KIT, PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER if the seal is broken or appears to have been tampered with.
- If ESTECEM causes an allergic reaction or oversensitivity, discontinue its use immediately.
- Use examination gloves (plastic, vinyl or latex) at all times when handling ESTECEM to avoid the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If the PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- Avoid contact of the PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER with eyes, mucosal membrane, skin and clothing.
 - If the PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
 - If the PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER comes in contact with the skin or clothing, immediately saturate the area with alcohol-soaked cotton swab or gauze.
- Instruct the patient to rinse his mouth immediately after treatment.

- ESTELINK or TOKUYAMA UNIVERSAL PRIMER should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- To avoid the unintentional ingestion of the PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER, DO not leave it unsupervised within the reach of patients and children.
- DO NOT expose ESTELINK or TOKUYAMA UNIVERSAL PRIMER or the vapor to open flame because ESTELINK or TOKUYAMA UNIVERSAL PRIMER is flammable.
 - When using a dispensing well supplied by other manufacturer.
 - DO TO avoid cross contamination of the PASTE, ESTELINK and TOKUYAMA UNIVERSAL PRIMER. Clean the dispensing well thoroughly with alcohol after each use.
 - DO NOT use ESTECEM directly to the cavity being in close proximity to the pulp. Pulp protection with glass ionomer lincet or calcium hydroxide is recommended. DO NOT use EUGENOL-BASED MATERIALS to protect the pulp as these materials will inhibit the curing of ESTELINK.
- When using a light-curing unit, protective eye shields, goggles or glasses should be worn.
- Be aware that when a restoration is shipped from misaligned occlusion or bruxism (clenching, grinding or tapping), the repaired restoration may chip again.
- DO NOT mix the PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER with other dental cements or bonding agents.
- The PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER should be brought to room temperature before using. After removing from the refrigerator, allow the product to sit for 20 minutes or until it reaches room temperature.

■PRECAUTIONS FOR PASTE

- Use the mixing-tip included in this KIT when extruding the mixed PASTE.
- Use a clean instrument or explier for the application or removal of PASTE.
- To avoid unintended hardening of the PASTE, after each use remove the mixing-tip from the automix syringe. Using a clean cloth or tissue wipe the excess paste from the tip of syringe prior to recapping the syringe using the cap provided with the syringe at the time of purchase.

■PRECAUTIONS FOR ESTELINK

- To avoid splashing the liquid, the bottles of ESTELINK should not be opened immediately after taking out of the refrigerator.
- Use the rubber dispensing well provided when dispensing and mixing of ESTELINK. If ESTELINK is not dispensed into the rubber dispensing it may be difficult to remove the cured ESTELINK from dispensing well. The cured ESTELINK can be removed by inverting the rubber.
- Some materials and medicaments (hemostatic material) inhibit setting/adhesion of ESTELINK for an extended period even after careful meticulous cleansing with water. DO NOT USE products which contain:
 - eugenol,
 - aluminum chloride,
 - zinc phosphate,
 - calcium sulfate,
 - aluminum sulfate,
 - diammine silver fluoride (molecular formula: Ag(NH₂)₂F),
- ESTELINK does not adhere to tooth structure immediately after applications of materials which contain:
 - hydrogen peroxide (oxydol),
 - sodium hypochlorite
- Be aware that the use of these materials on the day of cementation, however these materials can be used for root canal treatment as the inhibition caused by these materials usually evaporates within five days.
- Be sure to hold the bottles of ESTELINK BOND A and BOND B vertically when dispensing. Do not hold the bottles horizontally while dispensing, this could cause the primer to flow back into the nozzle and become contaminated.

■PRECAUTIONS FOR UNIVERSAL PRIMER

- To avoid splashing of the liquid, the bottles of TOKUYAMA UNIVERSAL PRIMER should not be opened immediately after taking out of the refrigerator.
- Be sure to hold the bottles of TOKUYAMA UNIVERSAL PRIMER A and B vertically when dispensing. Do not hold the bottles horizontally while dispensing, this could cause the primer to flow back into the nozzle and become contaminated.

■STORAGE

- Store under refrigeration at temperatures between 10 to 10° C (32 to 50°F).
- Keep away from heat, direct sunlight, sparks and open flames.
- DO NOT use after the indicated expiration date on the syringe/bottle/package.

■DISPOSAL

- Dispose of the PASTE, extruded unused paste from the automix syringe, mix and allow paste to harden before disposal.
- Unused ESTELINK and TOKUYAMA UNIVERSAL PRIMER is to be absorbed into an inert absorbent material such as gauze or cotton, and disposed of in accordance with local regulations.

■CLINICAL PROCEDURES

1. Tooth Preparation

Remove contamination from the tooth surface such as plaque, temporary cement residue, oil from materials to test crown fit, oil mist from a hand-piece, saliva, blood and exudate fluids by brushing with flour pumice, ultrasonic scaling or cleaning the surface of the tooth with water and the scrubber.

- If there is an adherent surface including uncut enamel, apply etching agent to the uncut enamel.
- DO NOT excite the vital tooth. Desiccation can lead to post-operative sensitivity.

2. Restoration Preparation

Follow the procedure shown below, depending on the material to be restored.

- Ceramics other than Porcelain, and Composite materials
 - Roughen the interior of the restoration by sandblasting (0.1 to 0.2MPa), air abrasion or grinding with diamond burs using a slow speed to prepare the surface for adhesion or thoroughly rinse and dry the surface.
- Porcelain
 - DO NOT rough the surface. Prepare the interior of the restoration, providing a clean and fresh surface for adhesion. Thoroughly rinse and dry the surface.
- Metal restorations
 - Roughen the area by sandblasting (0.3 to 0.5MPa), air abrasion or grinding with a diamond burs using a slow speed to prepare the surface for adhesion. Thoroughly rinse and dry the surface.

- When using the sandblaster or air abrasion, use at least 30 to 50 micron grid of alumina powder.
- Unable to roughen or prepare the adhesive surface, etch the area for adhesion with phosphoric acid then thoroughly rinse and dry.
- The substances listed below inhibit adhesion and must be removed from the adhesive surface by mechanical cleaning or alcohol soaking. Clean the restoration after treatment is complete.
 - Gypsum and investment
 - Silicone oil from fit check material
 - Oil mist from handpiece
 - Plaque, tartar, saliva, blood and exudate fluids
 - If necessary, protect the surface not being bonded with metal separator, paraffin film, Vaseline, etc.

- When using the sandblaster or air abrasion, use at least 30 to 50 micron grid of alumina powder.
- Unable to roughen or prepare the adhesive surface, etch the area for adhesion with phosphoric acid then thoroughly rinse and dry.
- The substances listed below inhibit adhesion and must be removed from the adhesive surface by mechanical cleaning or alcohol soaking. Clean the restoration after treatment is complete.
 - Gypsum and investment
 - Silicone oil from fit check material
 - Oil mist from handpiece
 - Plaque, tartar, saliva, blood and exudate fluids
 - If necessary, protect the surface not being bonded with metal separator, paraffin film, Vaseline, etc.

3. Restoration Pretreatment with the TOKUYAMA UNIVERSAL PRIMER

- Dispensing
 - Allow TOKUYAMA UNIVERSAL PRIMER to reach room temperature prior to use. Open the bottle by removing the cap and dispense one drop each of TOKUYAMA UNIVERSAL PRIMER A and B into the dispensing well. Mix thoroughly with a disposable applicator.
 - Complete the application within 5 minutes after dispensing since TOKUYAMA UNIVERSAL PRIMER contains volatile solvents.
 - Wipe off excess primer from the tip of the nozzle prior to closing.
 - Immediately after dispensing, tightly replace the bottle caps of TOKUYAMA UNIVERSAL PRIMER A and B.
- Application
 - Using the disposable applicator, apply the mixed primer on the surface to be bonded and wait 10 seconds.
 - Ensure primer covers all surfaces where the adhesive will be applied.
 - DO NOT over apply. One or two applications of TOKUYAMA UNIVERSAL PRIMER are sufficient. Too thick of a layer may result in poor adhesion.
 - DO NOT allow saliva, blood, oil, water etc. to come in contact with the surface to be bonded. If the treated surface is contaminated, thoroughly rinse with water, dry and re-apply primer.

- Air dry
 - Using an oil-free air/water syringe, apply mild air to the surface. Use a vacuum aspirator to prevent spatter of primer.
 - Be sure to dry the surface after TOKUYAMA UNIVERSAL PRIMER is applied. Incomplete drying may result in poor adhesion.

tooth (Adherent surface) Pretreatment with the ESTELINK

- If the adherent surface includes ceramic, porcelain, composite, and/or metal, apply TOKUYAMA UNIVERSAL PRIMER and dry prior to the application of ESTELINK.
 - 1)Dispensing
 - Allow ESTELINK to reach room temperature prior to use. Open the bottle by removing the cap and dispense one drop each of ESTELINK BOND A and BOND B into the dispensing well. Mix thoroughly with a disposable applicator.
 - Complete the application within 1 minute after dispensing since ESTELINK contains volatile solvents. After mixing, the color of ESTELINK changes gradually based on the reaction of the catalyst of ESTELINK, this is normal and does not indicate a problem with ESTELINK.
 - Wipe off excess ESTELINK from the tip of the nozzle prior to closing.
 - Immediately after dispensing, tightly replace the bottle caps of ESTELINK BOND A and BOND B.

- Application
 - Using the disposable applicator, apply the mixed ESTELINK on the surface to be bonded and wait 10 seconds.
 - Ensure ESTELINK covers all surfaces where the PASTE will be applied.
 - If the ESTELINK pools on the cavity floor or a cavosurface angle and is too thick to air thin, remove the excess with a paper point or a new disposable applicator after waiting for 2 seconds.
 - Be careful not to allow excess ESTELINK to flow into the subgingival. If ESTELINK flows into the subgingival, thoroughly rinse and dry the area.
 - DO NOT wait too long after the application of ESTELINK; this will lead to the loss of the initial wetting period. Finisher in utilizing an soufflé d'air forte sur la surface.

- If the adherent surface includes ceramic, porcelain, composite, and/or metal, apply TOKUYAMA UNIVERSAL PRIMER and dry prior to the application of ESTELINK.
 - 1)Dispensing
 - Allow ESTELINK to reach room temperature prior to use. Open the bottle by removing the cap and dispense one drop each of ESTELINK BOND A and BOND B into the dispensing well. Mix thoroughly with a disposable applicator.
 - Complete the application within 1 minute after dispensing since ESTELINK contains volatile solvents. After mixing, the color of ESTELINK changes gradually based on the reaction of the catalyst of ESTELINK, this is normal and does not indicate a problem with ESTELINK.
 - Wipe off excess ESTELINK from the tip of the nozzle prior to closing.
 - Immediately after dispensing, tightly replace the bottle caps of ESTELINK BOND A and BOND B.

- Application
 - Using the disposable applicator, apply the mixed ESTELINK on the surface to be bonded and wait 10 seconds.
 - Ensure ESTELINK covers all surfaces where the PASTE will be applied.
 - If the ESTELINK pools on the cavity floor or a cavosurface angle and is too thick to air thin, remove the excess with a paper point or a new disposable applicator after waiting for 2 seconds.
 - Be careful not to allow excess ESTELINK to flow into the subgingival. If ESTELINK flows into the subgingival, thoroughly rinse and dry the area.
 - DO NOT wait too long after the application of ESTELINK; this will lead to the loss of the initial wetting period. Finisher in utilizing an soufflé d'air forte sur la surface.

- Application
 - Using the disposable applicator, apply the mixed ESTELINK on the surface to be bonded and wait 10 seconds.
 - Ensure ESTELINK covers all surfaces where the PASTE will be applied.
 - If the ESTELINK pools on the cavity floor or a cavosurface angle and is too thick to air thin, remove the excess with a paper point or a new disposable applicator after waiting for 2 seconds.
 - Be careful not to allow excess ESTELINK to flow into the subgingival. If ESTELINK flows into the subgingival, thoroughly rinse and dry the area.
 - DO NOT wait too long after the application of ESTELINK; this will lead to the loss of the initial wetting period. Finisher in utilizing an soufflé d'air forte sur la surface.

- Application
 - Using an oil-free air/water syringe, apply mild air continuously to the ESTELINK surface. Use a vacuum aspirator to prevent spatter of primer.
 - Use a vacuum aspirator to prevent spatter of the primer.
 - If saliva, blood, or other fluids contaminate the applied ESTELINK, thoroughly rinse with water, dry and re-apply fresh ESTELINK.

5. Cementation and Final Bonding

- Mixing and Applying PASTE A and B
 - Before each use, remove the cap from automix syringe, and squeeze out a small quantity of paste A and B, making sure equal amounts are being dispensed through the two outlets of the syringe.
 - Mix thoroughly.
 - How to remove the cap: turn the cap 1/4 of a turn counterclockwise to align the projections on the cap with the grooves in the syringe. Holding the base of the cap, rotate the cap by twisting and pressing downward.
 - DO NOT allow PASTE A to come into contact with PASTE B, there is a possibility of hardening of the PASTE at the tip of syringe. Wipe the excess PASTE from the tip of syringe with a clean cloth or tissue before using a new mixing tip.
 - Do not mix paste A and B together in the syringe. (A and B) from the syringe and apply the mixed PASTE to the surface of the restoration)
 - If equal amounts of PASTE A and B are not dispensed, there is a possibility of poor polymerization. Ensure equal amounts are being dispensed through the two outlets of the syringe prior to attaching a mixing tip.
 - How to attach the mixing tip: align the projections on the adapter with the grooves in the syringe as closely as possible, and turn the mixing tip 1/4 of a turn clockwise to lock it in place.
 - Be sure to attach a mixing tip to the syringe when applying the PASTE to the surface of the restoration.
 - After use, remove the mixing tip from automix syringe and replace the cap of the automix syringe.
- Cementation of Restoration
 - Place the restoration on the tooth with firm pressure. Any excess PASTE at the margin can be removed with a brush tip.
 - DO NOT apply the PASTE to the adherent surface treated with ESTELINK to avoid unintended setting of the restoration. Especially in the case of cementing posts, never place the PASTE in the root canal. Be sure to apply the PASTE to the post and then place into the root canal. ESTELINK will accelerate the setting of the PASTE.
 - To keep the appropriate film thickness, seat the restoration as soon as possible after the application of the PASTE, and do not touch the restoration, if not there is a possibility that the restoration will not seat properly with cementation.

6. Removal of Excess PASTE.

There are two methods for removal of excess PASTE.

- When using the light-cure method
 - Once the restoration is seated, light-cure the excess PASTE with a dental curing light for 2 to 4 seconds. The gel state of the excess PASTE can be easily removed.
 - Excess light curing will lead to difficult removal of the excess cement.
- When using the self-cure method
 - The gel state of the excess cement can be easily removed within 1 to 3.5 minutes after seating.
 - Waiting more than 3.5 minutes after seating will lead to difficult removal of the excess cement.
 - The setting time of the PASTE may change (approximately 20 seconds at the maximum) depending on the oral temperature in each clinical case.
- Final Adjustment
 - Final PASTE hardening
 - a) In case of translucent restoration materials such as ceramics or composite materials. Irradiate the light to the applied PASTE on the seated restoration for 20 seconds or more. When the adherent area is too large to be completed by a single light-cure, multiple cures are recommended.
 - Confirm that the light-curing unit has sufficient intensity (>500mW/cm²) before using. Note that using a cracked light guide will lower the intensity.
 - In case of non translucent restoration materials such as metal.
 - Light-cure alone for 20 seconds or more, then allow the PASTE to set for 3 minutes with the patient applying normal occlusal pressure to the restoration.
 - Finishing
 - Check patient's bite, if necessary. Re-polish the restoration with material of choice.

IMPORTANT NOTE: The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure that this product is suitable for an appropriate application before use.

- If the patient complains of any discomfort, stop the application immediately. When the product specifications change, the instructions and precautions may also change.

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