Clinical Procedure Guide for

TOKUYAMA SHIELD FORCE PLUS

- · Read instruction before operation.
- Vor der ANwendung die Gebrauchsanweisung lesen.
- Leggere le istruzioni prima dell'uso.
 Veuillez vous référer au mode d'emploi
- avant utilisation.
- Lea las instrucciones antes del uso. Lees de aanwijzingen voor gebruik.

Treatment of dentinal hypersensitivity / Coating of exposed cervical dentin



Apply TOKUYAMA SHIELD FORCE Dispense PLUS, then leave for 10+sec



Apply weak air (5sec) → Strong air (5+sec)



Light cure for 10+sec

Sealing of prepared tooth to alleviate / prevent tooth sensitivity (Direct Restorations)



Dispense



Apply TOKUYAMA SHIELD FORCE Apply weak air (5sec) PLUS, then leave for 10+sec



→ Strong air (5+sec)



Light cure for 10+sec



* If using a total etch adhesive system, etchant must be applied "AFTER" TOKUYAMA SHIELD FORCE PLUS procedure.

TOKUYAMA SHIELD FORCE PLUS



Sealing of prepared tooth to alleviate / prevent tooth sensitivity (Indirect Restorations)




Dispense



Apply TOKUYAMA SHIELD FORCE PLUS, then leave for 10+sec



Apply weak air (5sec) → Strong air (5+sec)

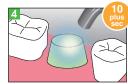


Light cure for 10+sec



Wipe unpolymerized layer with alcohol





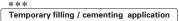



** BEFORE taking impression, it is essential to wipe dentin surface with alcohol, removing unpolymerized layer of TOKUYAMA SHIELD FORCE PLUS that could prevent an accurate impression. *** Do NOT use temporary resin-based material, it may adhere to the cured TOKUYAMA SHIELD FORCE PLUS. The use of water-based cement would be recommended for temporary cementing, and temporary stopping would be recommended for temporary filling.