# TOKUYAMA CUREGRACE

## ENGLISH

**Carefully read all of these instructions before using TOKUYAMA CUREGRACE.**

### PRODUCT DESCRIPTION AND GENERAL INFORMATION

- TOKUYAMA CUREGRACE is a self-curing acrylic resin, consisting of a POWDER and a LIQUID.
- The POWDER contains Methyl Methacrylate – Ethyl Methacrylate copolymer and Benzoyl peroxide. The LIQUID contains Methyl Methacrylate monomer.

### INDICATIONS

- Temporary inlay, crowns and bridges
- Repair of broken or cracked dentures
- Denture border extension
- Replacement of lost denture teeth
- Adjustment of the occlusal height of resin teeth

### CONTRAINDICATIONS

TOKUYAMA CUREGRACE contains methacrylic monomer. DO NOT use TOKUYAMA CUREGRACE for patients allergic or hypersensitive to methacrylic and related monomers or to any of the other ingredients.

### PRECAUTIONS

- TOKUYAMA CUREGRACE is designed for sale and use by licensed dental care professionals only, it is not designed for sale nor is it suitable for use by non-dental care professionals.
- Use TOKUYAMA CUREGRACE only as instructed herein. DO NOT use TOKUYAMA CUREGRACE for any application other than indicated.
- DO NOT use TOKUYAMA CUREGRACE if the safety seal is broken or if it appears the contents have been tampered with.
- LIQUID contains methyl methacrylate, which is a highly flammable and volatile solvent. DO NOT expose the LIQUID to open flame. Use in a ventilated room during use and close the cap tightly after use.
- If patient experiences an allergic reaction or sensitivity to TOKUYAMA CUREGRACE, remove the prosthesis and discontinue its use immediately.
- Use examination gloves (plastic, vinyl or latex) at all times while handling TOKUYAMA CUREGRACE to avoid the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If TOKUYAMA CUREGRACE comes in contact with the examination gloves, remove and dispose of the examination gloves and wash hands thoroughly with water as soon as possible.
- A polymerization exothermal reaction will occur after mixing the POWDER and the LIQUID. When TOKUYAMA CUREGRACE is seated intraorally, carefully monitor curing to prevent overheating. Remove from mouth before exothermic reaction starts to avoid burning or pulpal necrosis.
- Immerse hardened TOKUYAMA CUREGRACE in the water to reduce residual monomer before placing in the patient’s mouth.
- Avoid contact of TOKUYAMA CUREGRACE with eyes, skin and clothing.
  - In the case of contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
  - In the case of contact with the skin or clothing, immediately saturate the area with an alcohol soaked cotton swab or gauze.
  - Instruct the patient to rinse his/her mouth immediately after treatment.
- Whenever possible, avoid contact of TOKUYAMA CUREGRACE with oral mucous membrane other than the region to be lined. Instruct patient to rinse his/her mouth with adequate amount of water after the treatment.
- DO NOT allow excess materials to flow down the patient’s trachea or esophagus. Aspirated or ingested TOKUYAMA CUREGRACE can cause asphyxiation or other injury. TOKUYAMA CUREGRACE is not radiopaque, and radiographs will not detect aspirated or ingested material.
- DO NOT mix TOKUYAMA CUREGRACE with any materials not noted in these instructions.
- Always clean the spatula, brushes and cups with alcohol after each use to avoid cross contamination.
- To avoid inhaling dust while finishing, always wear a mask recommended by federal and/or local regulations.
- Read and follow the manufacturers’ instructions for all products mentioned in this instruction manual that are not included in this material.

### NOTES FOR USE

- Close the cap tightly, after use POWDER or LIQUID.
- DO NOT pour unused POWDER or LIQUID from the cup (ex. rubber cup) back into the bottle.
- DO NOT directly place a brush into the bottle of POWDER or LIQUID.
- DO NOT leave POWDER or LIQUID in the cup for a prolonged period of time.

### STORAGE

- Store TOKUYAMA CUREGRACE at a temperature between 0 – 25 °C (32 – 77 °F).
- Avoid direct exposure of TOKUYAMA CUREGRACE to heat, open flame, humidity and sunlight.
- DO NOT use TOKUYAMA CUREGRACE after the indicated date of expiration.
- DO NOT leave TOKUYAMA CUREGRACE unnecessarily within the reach of patients and children to avoid being swallowed by mistake

### DISPOSAL

- To dispose of materials, mix the POWDER and LIQUID together, and allow it to harden. Take care of the exothermic reaction produced by the setting material.

### CLINICAL APPLICATION OF TOKUYAMA CUREGRACE

#### (1) Dough Method

- Measuring and mixing
  - The standard mixing ratio is 2g of POWDER to 1mL of LIQUID. Use approximately 2-graduations of the POWDER in the measuring cup for 1-graduation of LIQUID (marked “1”) in the dropper.
  - Dispense desired amount of liquid into the rubber cup then add the powder to the liquid and mix gently with the spatula for about 10-15 seconds.
  - Note:
    - Put the LIQUID in the cup first and then the POWDER to minimize the formation of bubbles.
    - POWDER/LIQUID ratio is adjustable between 1.5g/1mL and 2.0g/1mL.
- Application of the resin
  - With the spatula, apply the dough-like resin to the surface of the treated area
  - Note:
    - All manipulations should be completed before 1.5 minutes after mixing when curing process starts.
- Adjusting and polishing
  - After final hardening, use a carbide bur, silicone point, finishing wheel, etc. to adjust and polish the prosthesis.

#### (2) Brush-on Technique

- Preparing
  - Dispense the desired amount of POWDER and LIQUID into separate cups.
- Making the mixture
  - Moisten the brush tip in the liquid, and dip the brush tip into powder. Some powder will stick to the brush and curing starts.
- Application of the resin
  - Brush the mixed material onto the desired surface.

- Patterning or repairing
  - Pattern or repair by repeating the above procedure (2-3).
  - Note:
    - Clean the brush quickly after use to avoid material hardening on the brush.

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